

# Conductive Gel With Aloe Vera

## UP240 3.00 Oz. Tube

For use with electrotherapy treatment. This conductive gel can be used to recondition re-useable electrodes allowing the electrodes to maintain conductive properties after several uses. **THIS PRODUCT WILL ALLOW THE ELECTRODE TO CONDUCT AS WELL ON DAY (7) OR LONGER AS ON DAY ONE. THIS EXTENDS ELECTRODE LIFE UP TO 4X LONG.**

Electrodes also need TAC gel added towards the end of use so they adhere or maintain an ability to stick to skin surface.

### Directions:

Add a small dab to the gel side of electrode, rub into the electrode, place electrode back onto plastic backing and allow to absorb. Do not over use gel on an electrode. Do this several times using only a small amount each time until the electrode is replenished.



**Manufactured by Uni-Patch Medical Supplies  
Founded 1978**